Thermometer easy

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EN

Important safety instructions

- The thermometer is suitable exclusively for measurement of the bodytemperature!
- The minimum measurement time until the signalling tone [beep] must be maintained without exception!
- The thermometer contains small parts (battery, etc.) that can be swallowed by children. Therefore never leave the thermometer unattended to children.
- Protect the thermometer from hitting and striking!
- Avoid ambient temperatures above 60 °C. **Never** boil the thermometer!
- The manufacturer recommends to verify the accuracy by an authorised laboratory every 2 years.

 Batteries and electronic instruments must be disposed of in accordance with the locally applicable regulations, not with domestic waste.

Turning on the thermometer

To turn on the thermometer, push the button over the display window; a short signalling tone [beep] signals «thermometer ON». A display test is performed. All segments should be displayed. Then at an ambient temperature of less than 32 °C, an «L» and a flashing «°C» appear at the upper right of the display. The thermometer is now ready for use.

Function test

Proper functioning of the thermometer is tested automatically each time it is turned on. If a malfunction is detected (measurement inaccuracy), this is indicated by «ERR» on the display, and a measurement becomes impossible. In this case, the thermometer must be replaced.

Using the thermometer

Choose the preferred method. During a measurement, the current temperature is displayed continuously. The «°C» symbol flashes during this period of time. If the signalling tone sounds (beep-beep-beep-beep 10 times) and the «°C» is no longer flashing, this means that the measured increase in temperature is now less than 0,1 °C in 16 seconds and that the thermometer is ready for reading. To prolong the life of the battery, turn it off after use by pressing the operating button briefly. Otherwise the thermometer will automatically turn off after about 10 minutes.

Storage of measured values

If you hold the button down for more than 3 seconds when turning the thermometer on, the maximum temperature stored automatically at the last measurement will be displayed. At the same time, a small «M» for memory will appear at the right of the display. After 3 more seconds, this value will disappear and the thermometer will switch to the normal measurement mode. Then the stored value is erased.

Measuring methods

In the mouth (oral)

Position the thermometer in one of the two pockets under the tongue, to the left or right of the root of the tongue. The measuring sensor must be in good contact with the tissue. Close your mouth and breath evenly through the nose to prevent the measurement from being influenced by inhaled/exhaled air. Approx. measuring time: 1 minute!

In the anus (rectal)

This is the most reliable measuring method, and is especially suitable for infants and small children. Carefully insert the measuring sensor of the thermometer 2 to 3 cm into the anal aperture. Approx. measuring time: 1 minute!

In the armpit (axillary)

To receive more reliable results apornorm® by microlife recommend to measure temperature orally or rectally. A minimum measurement time of 3–5 minutes is recommended although the acoustic signal could ring out even earlier.

Cleaning and disinfecting

Name: Isopropyl alcohol 70 %;

Use a cotton swab moistened with alcohol (70 % Isopropyl) to clean and disinfect the thermometer casing and the measuring probe. Never use abrasive cleaning agents, thinners or benzene for cleaning and never immerse the instrument in water or other cleaning liquids.

Battery replacement

When the «▼» symbol (upside-down triangle) appears in the lower right of the display, the battery is exhausted and needs replacing. For the battery replacement please remove the compartment lid of the thermometer. Replace the exhausted battery by positioning the + at the top. Make sure a battery of the same type is on hand. Batteries can be purchased at any electrical goods shops. For battery disposal please refer to «safety instructions».

	Technical specifications
Type	Maximum thermometer
Measurement range	32,0 °C bis 43,9 °C Temp. < 32,0 °C: display «L» for low (too low) Temp. > 43,9 °C: display «H» for high (to high)
Measurement accuracy	±0,1 °C between 34 °C and 42 °C
Operating temperature	10 °C to 40 °C
Self-test	Automatic internal check at a test value of 37 °C; if there is a deviation of > 0,1 °C, «ERR» (error) is displayed
Display	Liquid crystal display (LCD) with three digits. Smallest unit of display: 0,1 °C
Signalling tone	For signalling that the thermometer is ready to use and that the temperature increase is less than 0,1 °C / 16 seconds
Memory	For storing the last measured value
Storage Environment	temperature: -25 °C to +60 °C, humidity: 15 % to 95 % (noncondensing)
Battery	1,5 / 1,55 V; LR 41
Reference to standards	EN12470-3, clinical thermometers, ASTM E 1112, IEC 60601-1, IEC 60601-1-2
Electromagnetic Compatibility	Device fulfills the stipulations of the standard IEC 60601-1-2

Guarantee

We grant you a warranty of 30 years after the date of purchase. Any damage caused by improper handling shall not be covered by the warranty. Batteries and packaging are also excluded from the warranty. All other damage claims excluded. A warranty claim must be submitted with the purchase receipt. Please pack your defective thermometer well and send with sufficient postage to:

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WEPA Apothekenbedarf GmbH & Co. KG, 56204 Hillscheid