POLYHYL®

Natriumhyaluronat

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Description Polyhyl® is a sterile, isotonic, pyrogen-free, viscoelastic preparation of highly purified sodium hyaluronate, dissolved in a buffered physiological solution (pH 6.8 to 7.5) for intraocular application. It is a clear solution in a ready-to-use syringe for single use. It is packaged in a sterile blister or edge-sealed pouch.

Characteristics Sodium hyaluronate is a biopolymer obtained from bacterial biofermentation that belongs to the class of glycosaminoglycans. It has a high molecular weight, is highly purified and possesses favorable physical and rheological properties for ophthalmic surgery. Hyaluronic acid is a biological and biocompatible substance. The viscoelastic properties of Polyhyl® allow lubrication, support and protection of ocular tissues during ophthalmic surgery. It maintains the depth of the anterior chamber. Polyhyl® forms a thin protective layer on the ocular cells and tissues and facilitates the insertion of the intraocular lens. Polyhyl® is free from preservatives. Due to its solubility in water, Polyhyl® can be easily removed by irrigation and aspiration at the end of surgery.

Indications Polyhyl® serves as an adjuvant for the following operations:

Anterior chamber surgery of the eye, and especially:

- Cataract surgery with or without intraocular lens implantation
- Glaucoma surgery

Contraindications Hypersensitivity against sodium hyaluronate.

Mode and duration of application If Polyhyl® was stored in the refrigerator, the syringe should be removed at least 20 minutes prior to use from the refrigerator. When the air has escaped from the ophthalmological Luer-Lock cannula, Polyhyl® can be carefully injected into the anterior chamber using the enclosed sterile single-use cannula or a comparable cannula. Dosage depends on the kind of surgery. To replace loss of viscoelastic by outflow or irrigation, several injections can be made. Remove Polyhyl® completely at the end of surgery using a suitable irrigation / aspiration device.

Precautions Observe all usual precautions required during ophthalmic surgery! Together with the enclosed sterile single-use cannula Polyhyl® is designed for single intraocular application only. Check the integrity of the primary package (sterile pack) prior to the use of Polyhyl® to guarantee sterility. After the intraoperative use of Polyhyl® a transient post-operative intraocular pressure increase may occur. A suitable therapy for reduction of the intraocular pressure decreases the pressure increase.

Interactions with other devices or drugs There is an incompatibility of sodium hyaluronate and quaternary ammonium compounds such as solutions containing benzalconium chloride. Therefore Polyhyl® must never get into contact with surgical instruments rinsed with these solutions or with ophthalmics containing quaternary ammonium compounds as preservatives.

Shelflife Do not use Polyhyl® after the expiration date

Storage Store Polyhyl® between 2° and 25°C protected from light. Do not freeze Polyhyl®. Avoid shock.

Note For use during surgery: Do not exert any more pressure on the plunger prior to removing the cannula from the eye. This is to avoid aspiration of air bubbles into the cannula.

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Polytech Domilens GmbH