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Akanni ahc-Hydrogel⁹⁰

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Instruction for Use

Akanni ahc-Hydrogel⁹⁰

Ready-to-use hydrogel for cleansing and moistening of wounds

Intended use:

Akanni ahc-Hydrogel⁹⁰ is used for cleansing, long-term moisturizing and keeping acute and chronic wounds moist. It forms a protective film, keeps the wound moist and thus promotes wound healing. Akanni ahc-Hydrogel⁹⁰ forms an ideal wound healing environment, supporting the natural wound healing process. Wound odors are quickly and reliably eliminated using Akanni ahc-Hydrogel⁹⁰.

Product properties:

Due to its properties, Akanni ahc-Hydrogel⁹⁰ can be applied well and quickly, without any additional aids, and remains as a hydrogel in the wound. The application of Akanni ahc-Hydrogel⁹⁰ is non-irritating, and virtually painless. Akanni ahc-Hydrogel⁹⁰ is suitable for wounds on skin and mucosa. There is no restriction on the frequency and duration of use. Residue of Akanni ahc-Hydrogel⁹⁰ may remain in the wound. As a multi-dose container, Akanni ahc-Hydrogel⁹⁰ is preserved with NaOCl/HOCl. This allows the use of the opened individual product for 12 weeks.

Application information: Shake the bottle well before use!

Before first application and at dressing changes, wound cleansing is recommended to remove exudate, fibrinoid deposits and other contaminants. Wound irrigation solutions and wound sprays based on NaOCl/HOCl are suitable for this purpose. Spray Akanni ahc-Hydrogel⁹⁰ with several bursts from a short distance until the wound is completely wetted, depending on its nature. The frequency of dressing changes depends on the nature of the wound and should

be adapted to the particular wound situation. Akanni ahc-Hydrogel⁹⁰ can be used in combination with common compresses, plasters and other common dressing materials. Please also refer to the instructions for use of the respective product. Akanni ahc-Hydrogel⁹⁰ can be used without restriction regarding age and gender of the patient. There are no known undesirable side effects. A slight chlorine odor is a sign of the activity of the preservative system. Akanni ahc-Hydrogel⁹⁰ can be used by non-professionals if they have been instructed by healthcare professionals.

Application restrictions:

Akanni ahc-Hydrogel⁹⁰ should not be used if there are known allergies to any of the ingredients. Do not use in combination with other wound cleaning sprays, solutions, gels and pre-saturated dressings. Do not use simultaneously with products containing chlorhexidine. Do not use Akanni ahc-Hydrogel⁹⁰ on bleeding or exuding wounds. Dressing materials should be compatible with oxidising agents, especially hypochlorite. For use during pregnancy, while breast-feeding and in children under two years of age, consult your doctor or pharmacist beforehand. In children up to 12 years of age, the application should be supervised by an adult. If there is redness, swelling or pain in the wound area or if the condition of the wound does not improve within a few days, a doctor should be consulted.

Safety instructions:

Do not swallow. Avoid contact with eyes. In case of eye contact, rinse with plenty of water. Use the product only until the indicated expiry date and within 3 months after opening. Please report a potentially serious incident involving the product immediately to Prontomed (vigilanz@prontomed.de) and the competent authority in your country.

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